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D005\_A0086\_210707

Please read these instructions for use carefully before using the product for the first time. Save this information. You may need to read it again.

### What is Vernivia® Vaginal Mousse used for?

- Treats bacterial vaginosis
- Prevents and treats bacterial imbalance
- Rapidly relieves embarrassing odour and discharge
- Inhibits the growth of harmful bacteria
- Increases the number of Lactobacillus
- Reinforces the vagina's own natural defence system
- Strengthens the resistance against future bacterial and yeast infections
- Restores the pH balance

Vernivia® Vaginal Mousse is used to treat a bacterial infection in the vagina, so-called bacterial vaginosis. Vernivia® Vaginal Mousse rapidly relieves common symptoms of bacterial vaginosis, such as abnormal discharge, unpleasant odour and irritation. Vernivia® Vaginal Mousse has a clinically proven effect.

### What is bacterial vaginosis?

Bacteria are always present in the vagina, both protective Lactobacillus and other bacteria. Lactobacillus produce lactic acid and hydrogen peroxide, which neutralise harmful bacteria and fungi. Bacterial vaginosis occurs when an imbalance has developed between the bacteria in the vagina, with the Lactobacillus no longer dominating.

Bacterial vaginosis is common among women of childbearing age. The most characteristic symptom is discharge with an unpleasant odour that is caused by harmful bacteria. The symptoms often get worse after periods or unprotected sex. Many women may therefore experience a change in their symptoms over time.

### How does Vernivia® Vaginal Mousse work?

The benefit of using Vernivia® Vaginal Mousse lies in the fact that it is applied to the entire vagina and so effectively treats the entire vaginal wall. Several of the ingredients used in Vernivia® Vaginal Mousse are found naturally in the vagina, such as lactic acid, hydrogen peroxide, water and air.

Vernivia® Vaginal Mousse rapidly neutralises the bacteria that cause the unpleasant discharge and also restores the pH balance. At the same time, Vernivia® stimulates the growth of protective Lactobacillus. Production of hydrogen peroxide and lactic acid by Lactobacillus plays an important part in the vagina's natural defence against bacterial and fungal infections.

### How do I use Vernivia® Vaginal Mousse?

Vernivia® Vaginal Mousse is applied in the vagina, at bedtime, for seven consecutive days. Do not discontinue treatment earlier than that, even if your symptoms clear up quickly.


If you have missed a dose, treatment should continue the next evening until you have had a total of seven doses.

Vernivia® Vaginal Mousse should not be used during menstruation. Avoid starting treatment in the days immediately before your next expected period. If your period does start during treatment, stop the treatment and start again from day 1 once the bleeding has decreased.

#### Instructions day 1

Test the applicator before using Vernivia® Vaginal Mousse for the first time:

1. Remove the top from the canister.
2. Attach the applicator carefully to the canister.
3. Direct the tip of the applicator down into the top.
4. Press the button until a small amount of mousse is dispensed.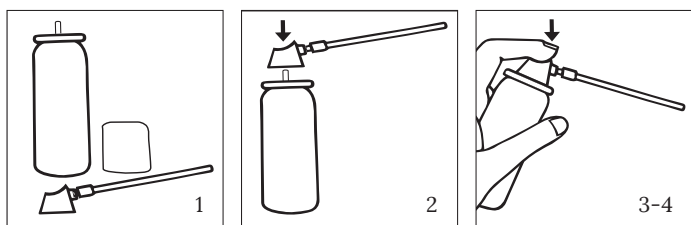


Do not use the product if the applicator is damaged or if no mousse is dispensed after repeated attempts.

#### Instructions day 1-7

Apply Vernivia® Vaginal Mousse at bedtime. To protect your bedlinen, you can place a towel beneath you.

1. Lie on your back with your knees drawn up slightly.
2. Carefully insert the applicator as far as you can into your vagina without it becoming uncomfortable.
3. Dispense the mousse by pressing the button for about one second.
4. After application, leave the applicator attached to the canister. Wipe the applicator if necessary.
5. The applicator can be rinsed with lukewarm water the next morning if necessary.

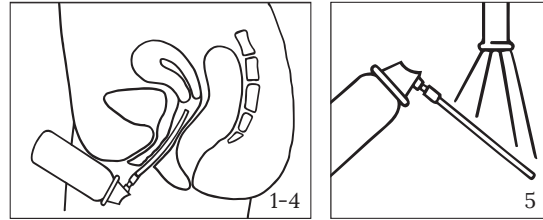After application, there may be some leakage of mousse from the vagina, which can be wiped away with a tissue. However, if there is a large amount of leakage, you should press the button for slightly less time when applying the next dose. You may find it helpful to use a pantiliner during the treatment. Contact your doctor if your symptoms persist after completing seven days of treatment with Vernivia® Vaginal Mousse.

#### Warnings

- The product is not recommended for use in children.
- Contact your doctor if you are pregnant or plan to become pregnant.

#### Precaution

- Do not use Vernivia® Vaginal Mousse at the same time as other vaginal products, such as soap, intimate douches, lubricants, spermicidal agents, vaginal creams or pessaries.
- As with all vaginal infections, you should avoid intercourse during the infection and during the period when you are using Vernivia® Vaginal Mousse.
- Do not use the product together with condoms or diaphragms, as Vernivia® Vaginal Mousse may have an impact on their effectiveness.
- Do not use Vernivia® Vaginal Mousse if you have a known allergy to any of the ingredients.
- Vernivia® Vaginal Mousse may cause some mild transient burning sensation on application. If this feels very uncomfortable, wash with lukewarm or warm water. Contact your doctor if the irritation persists.

#### What does the pack contain?

The pack contains a canister, instructions for use and an applicator developed for vaginal use. The applicator is reusable, thin and semi-flexible with a rounded tip.

Vernivia® Vaginal Mousse is free from antibiotics, perfume, parabens and hormones.

**Ingredients:** Lactic acid, glycerol monolaurate, glycerol mono-myristate, hydrogen peroxide, EDTA, sodium pyrophosphate, sodium stannate, sodium oxalate, sodium hydroxide and purified water.

#### How should Vernivia® Vaginal Mousse be stored?

The product should be stored out of the reach of children. Do not store the product above 30°C or below 2°C. Avoid storing the product in direct sunlight. Do not use the product after the expiry date or more than one month after opening the product.

#### How can Vernivia® Vaginal Mousse be sorted for recycling?

Contact your local authority for regional information on recycling.

**WARNING** Pressurised container: May burst if heated. Do not pierce or burn, even after use. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not expose to temperatures exceeding 50 °C/122 °F. Protect from sunlight. Keep out of reach of children.

D005\_A0086\_210707